




Replace key lock to Seal lock


Apollo cart - Solid or Glass doors

1 Removing the existing lock

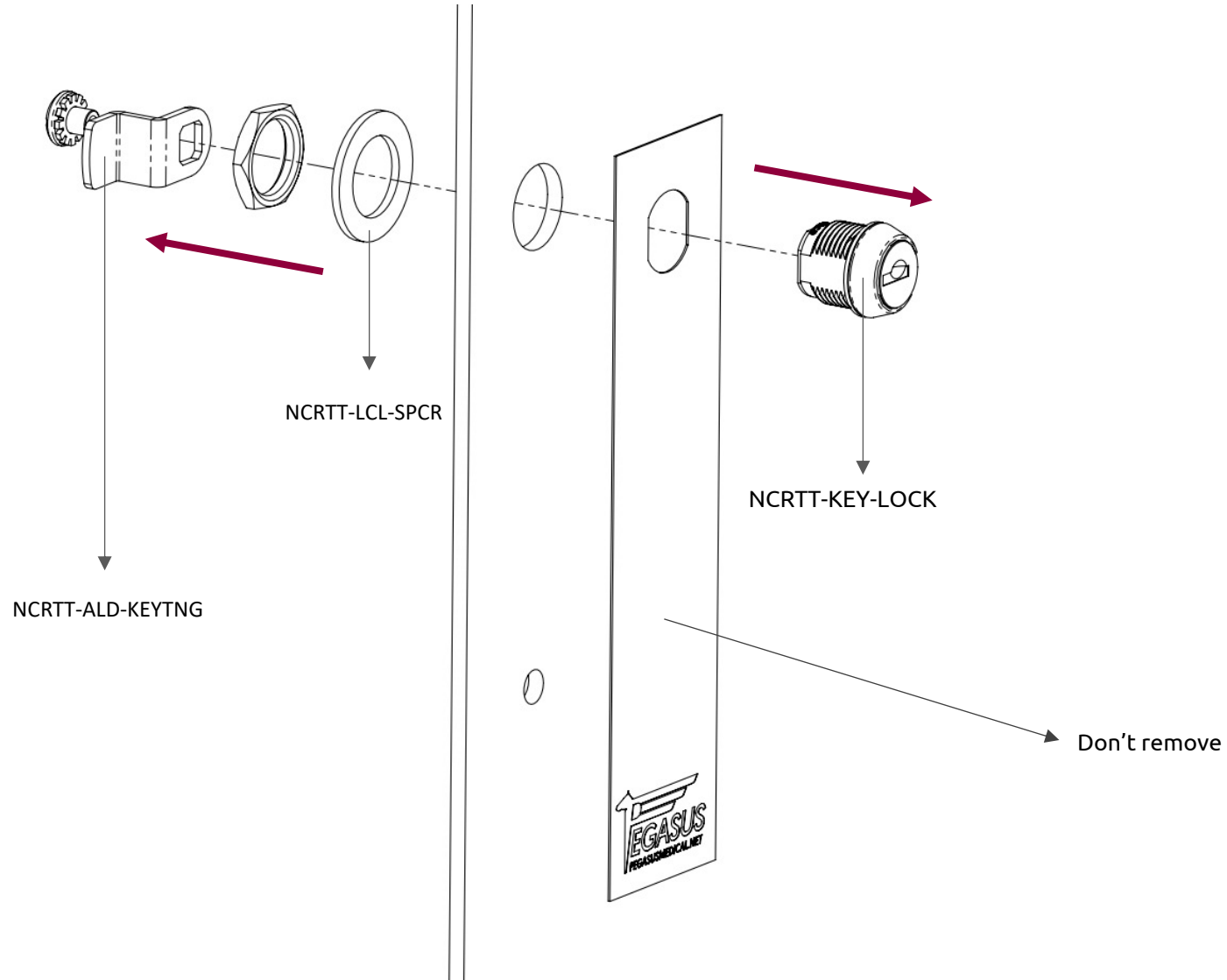


2 Remove the cover from the sticker

Remove the cover from the sticker

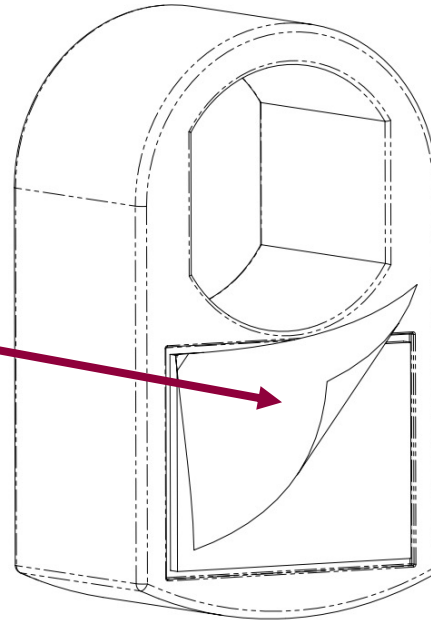
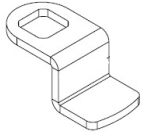
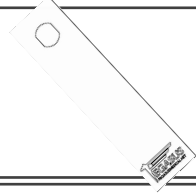
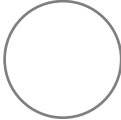
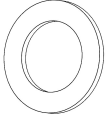
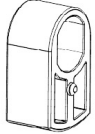
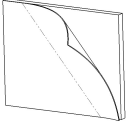


NCRTT-ALD-BRO-LOCK-SPCR

3 Installing the new lock

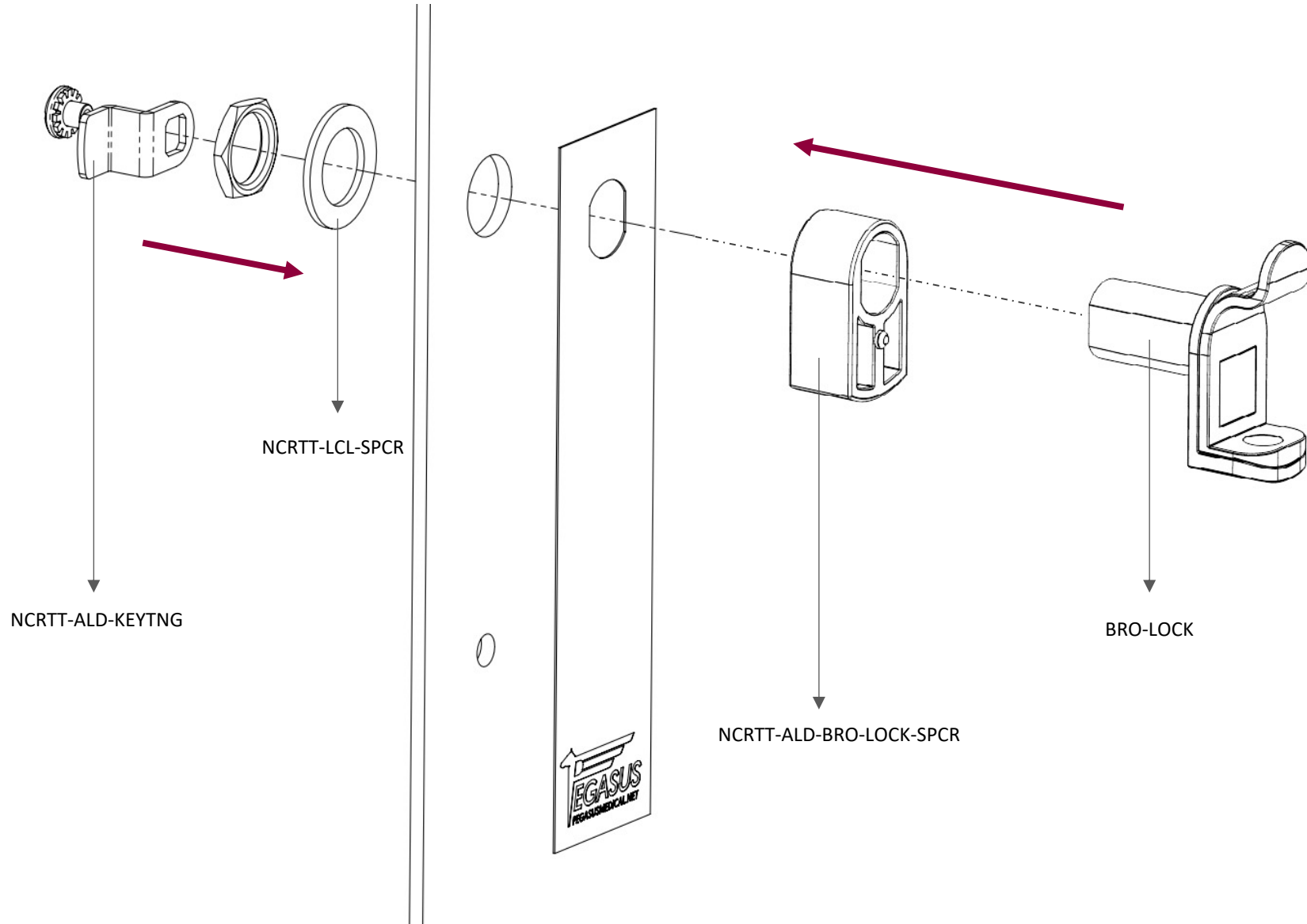

List of parts

	BRO-LOCK		NCR TT-ALD-KEYTNG		NCR TT-ALD-LOCKSPCR-V		NCR TT-STK-HOLCOV
	NCR TT-LCL-SPCR		NCR TT-ALD-BRO-LOCK-SPCR		NCR TT-AR-AT	Nielsen-keyWHole	



**For more information,
contact our customer service**

1-888-276-4750

info@pegasusmedical.net

www.pegasusmedical.net